

Optotune

Focus-tunable lenses for endoscopy

Zurich, July 2020

Dr. David Leuenberger, Senior Business Development Manager

Bernstrasse 388 | CH-8953 Dietikon | Switzerland Phone +41 58 856 3011 | www.optotune.com | info@optotune.com



Value proposition for endoscopes



- Ever higher resolution endoscopy have decreasing depth of field.
 - →A tunable lens allows to adjust the working distance from few mm to hundreds of mm.
- Current endoscopy systems with external cameras use a focusing/zoom rings that are hard to move with a single hand.
 - → Push button focusing allows to do single-handed refocusing
- Current endoscopy system do not feature an auto-focus
 - → Combining a tunable lens with a TOF distance sensor would provide real-time auto-focus.
- The high tuning speed of our lenses allow new functionalities:
 - Hyperfocal imaging through image stacking
 - 3D imaging from depth of focus






- Our value proposition
- Optical functions enabled by tunable lens
 - Accommodation in proximal camera
 - Accommodation for chip-on-a-tip
 - Extended depth of field, Depth from focusing, Macro
- Standard products
- Reliability



Tunable lens in camera adapter




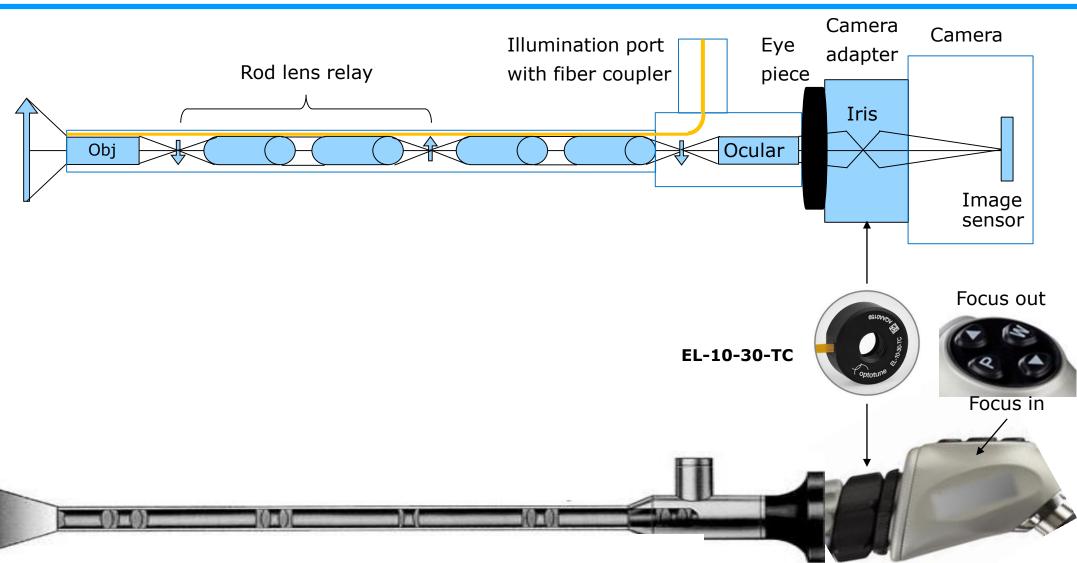



Example design for tunable camera adapter



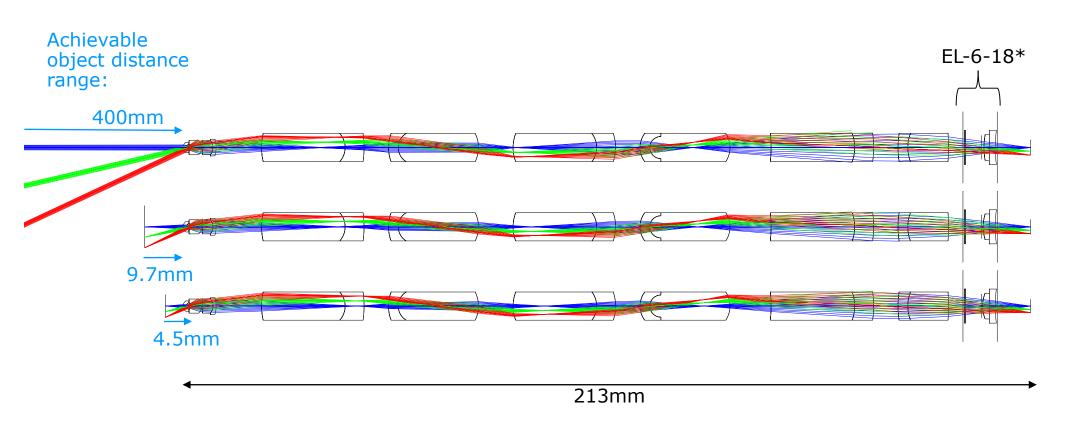


*assumed EL-6-18 tuning range: BFL -385 ... +55mm



Small tunable lens in ocular




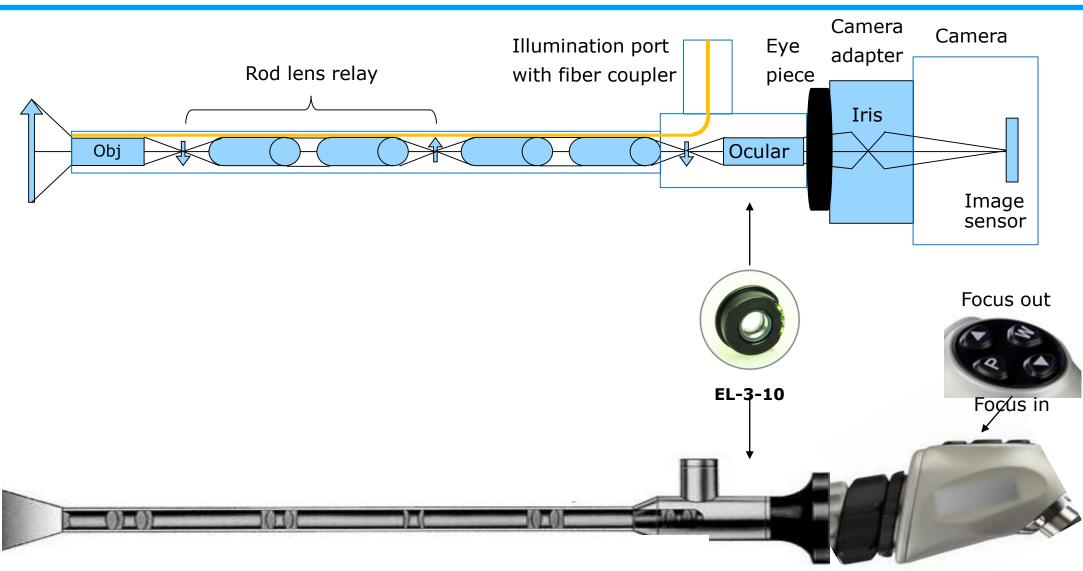





Example: Electrically tunable fluidic lens imaging system for laparoscopic fluorescence-guided surgery



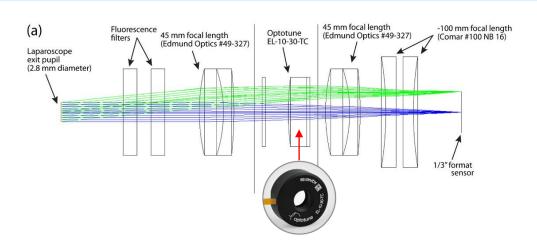


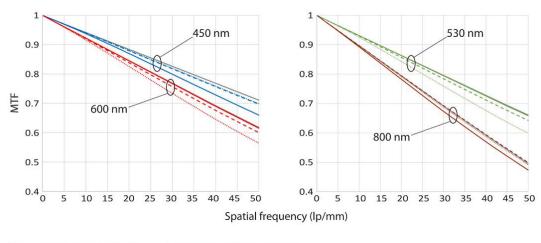


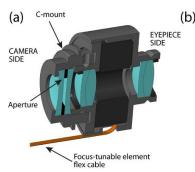







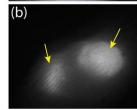

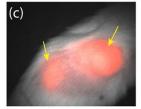



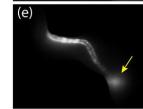

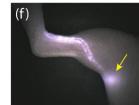

In vivo evaluation of laparoscopic imaging. (a) White reflectance, (b) fluorescence and (c) combined fluorescence + white reflectance images of a subcutaneous tumor (right arrow), obtained with the TLS. Fluorescence signal of the probe accumulating in the kidney is also shown (left arrow). (d) White reflectance, (e) fluorescence and (f) combined fluorescence + white reflectance images of the left inguinal lymph node (arrows)






- Our value proposition
- Optical functions enabled by tunable lens
 - Accommodation in proximal camera
 - Accommodation for chip-on-a-tip
 - Extended depth of field, Depth from focusing, Macro
- Standard products
- Reliability



Optotune's lens technology can be miniaturized

- Optotune has the concepts and technology to build electrically tunable lenses that fit into the tip of an endoscope
- Target specs: 1mm CA, ≈2.4 mm OD, 1-2 mm thickness and >= ±18 dpt tuning range
- Please contact <u>sales@optotune.com</u> to learn more about custom lens developments

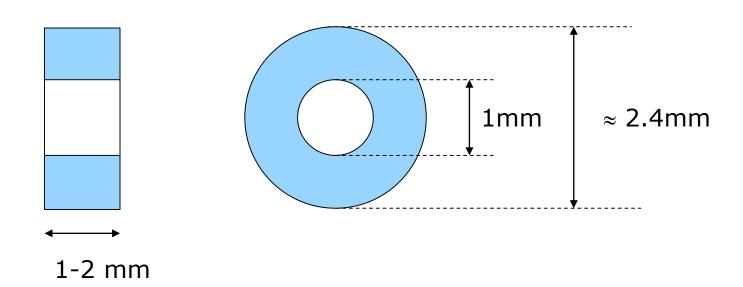






- Our value proposition
- Optical functions enabled by tunable lens
 - Accommodation in proximal camera
 - Accommodation for chip-on-a-tip
 - Extended depth of field, Depth from focusing, Macro
- Standard products
- Reliability



Depth from focus (DFF)



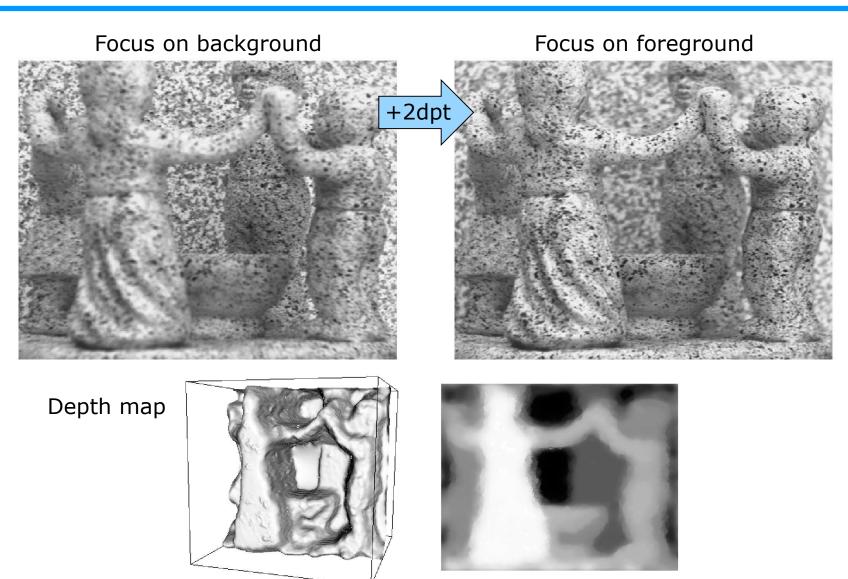






Focus stacking for "hyper-focus" images




Z-stack of e.g. 10 to 30 images*

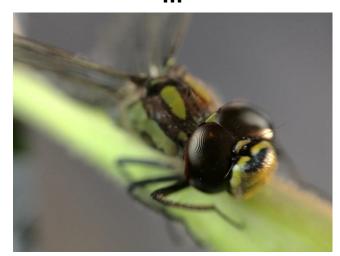

Rendered hyper-focus image**



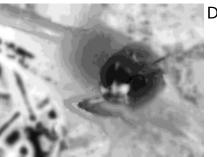

Depth map Video



^{*}Ideally the number of frames to acquire is = Z-range / DoF **Rendered with Helicon Focus 6.7.1 software from 15 pictures (offline)





- Our value proposition
- Optical functions enabled by tunable lens
 - Accommodation in proximal camera
 - Accommodation for chip-on-a-tip
 - Extended depth of field, Depth from focusing, Macro
- Standard products
- Reliability



Optotune's	electrically	focus	tunable	lenses



^{*} Depends on selected optical fluid ** vertical / horizontal optical axis





- Our value proposition
- Optical functions enabled by tunable lens
 - Accommodation in proximal camera
 - Accommodation for chip-on-a-tip
 - Extended depth of field, Depth from focusing, Macro
- Standard products
- Reliability



Auto-cleaving



• The EL-10-30 lens (in a sealed housing) has been tested by a 3rd party and survived 72 cycles of auto-cleaving (vapor, 140°C, 10 min).



Lifetime test of EL-10-30

1

Test	Test conditions	Status
Mechanical cycling	40 million full-range cycles (0 to 300 mA rectangular, at 10 Hz) 5 billion sinusoidal cycles at resonant frequency	Passed
High temperature test	85±2°C; rel. hum. <6% for 168 hours, non-operational	Passed
Temperature cycling test	-40°C / +85°C for 30 min each, 3 min transition time, 100 cycles	Passed
Damp heat cycling test	25°C / 55°C at 90-100% relative humidity, 3 hour transition time, 24h per cycle (9h plus transition time each), 18 cycles	Passed
Shock test:	800g for 1ms duration, 5 pulses in each direction (30 pulses in total)	Passed
Solar radiation test:	1120 W per m2 (IEC 60068-2-5), 8 h irradiation & 16 h darkness, 10 cycles	Passed



Polymer lenses can withstand high power!



- Only 0.1% absorption in VIS & NIR range
- The following tests were successful with EL-10-30 (LD fluid)
 - 1070 nm, 200 W CW on a 3 mm beam diameter (equivalent to 2.2 kW/cm²)
 - 1064 nm, 20 ns-pulsed at 50 kHz, 10 W average power on a 0.05 mm beam diameter (10 J/cm²)
 - 355 nm, 20 ns-pulsed at 50 kHz, 7 W average power on a 0.05 mm beam diameter (7 J/cm²)
 - 1064 nm, 12 ps-pulsed at 8.2MHz, 38 W average power on a 2 mm beam diameter (147 $\mu J/cm^2$)
 - 850 nm, 140 fs-pulsed at 80 MHz, 3 W average power on a 6 mm beam diameter (0.13 μJ/cm²)
 - 345 nm, 500 fs-pulsed at 200 kHz, 0.5 W average power on a 3 mm beam diameter (35 μJ/cm²)
- Customized cover glasses are available







shaping the future of optics

Optotune AG Bernstrasse 388 CH-8953 Dietikon Switzerland

Phone: +41 58 856 3000 | Fax: +41 58 856 3001

www.optotune.com | info@optotune.com